




I. Test Setup

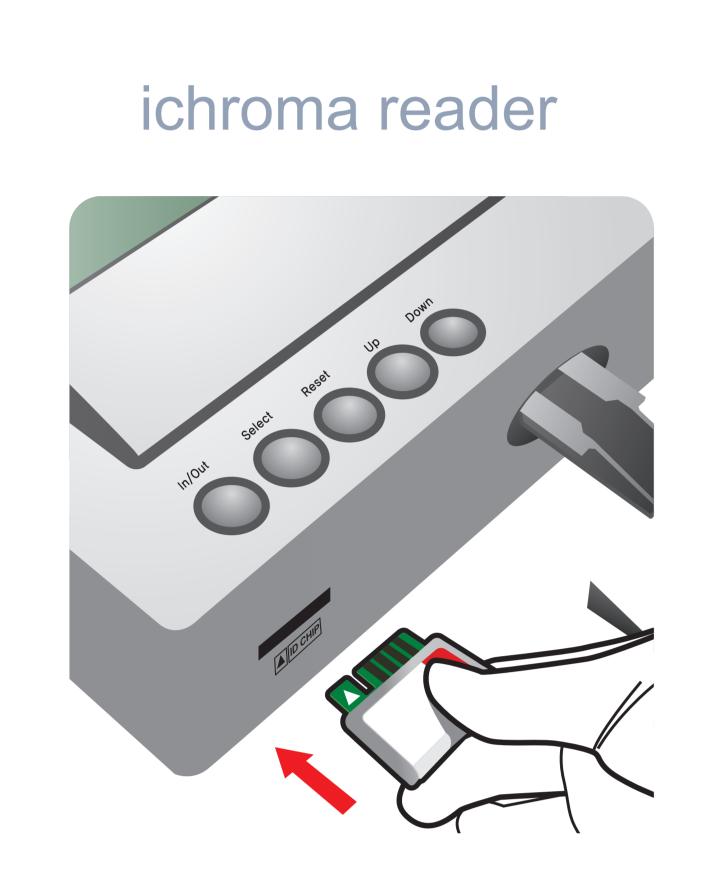


Insert ID Chip and press 'SELECT' button.

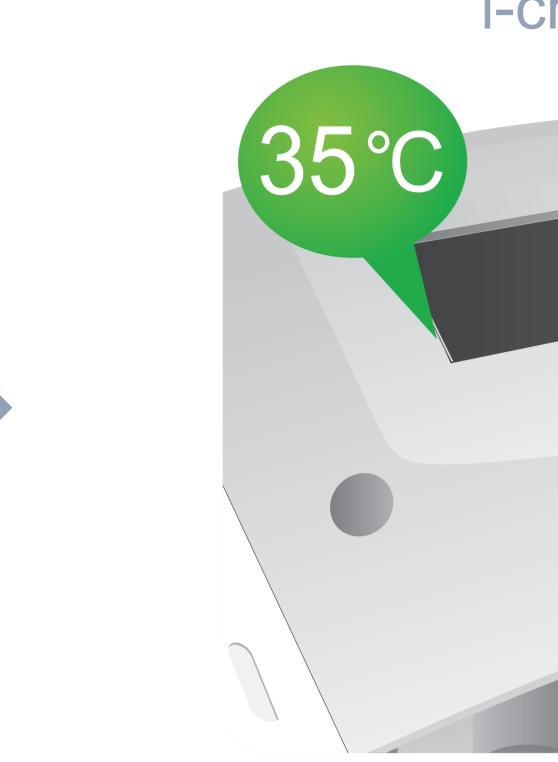

II. Mix Sample

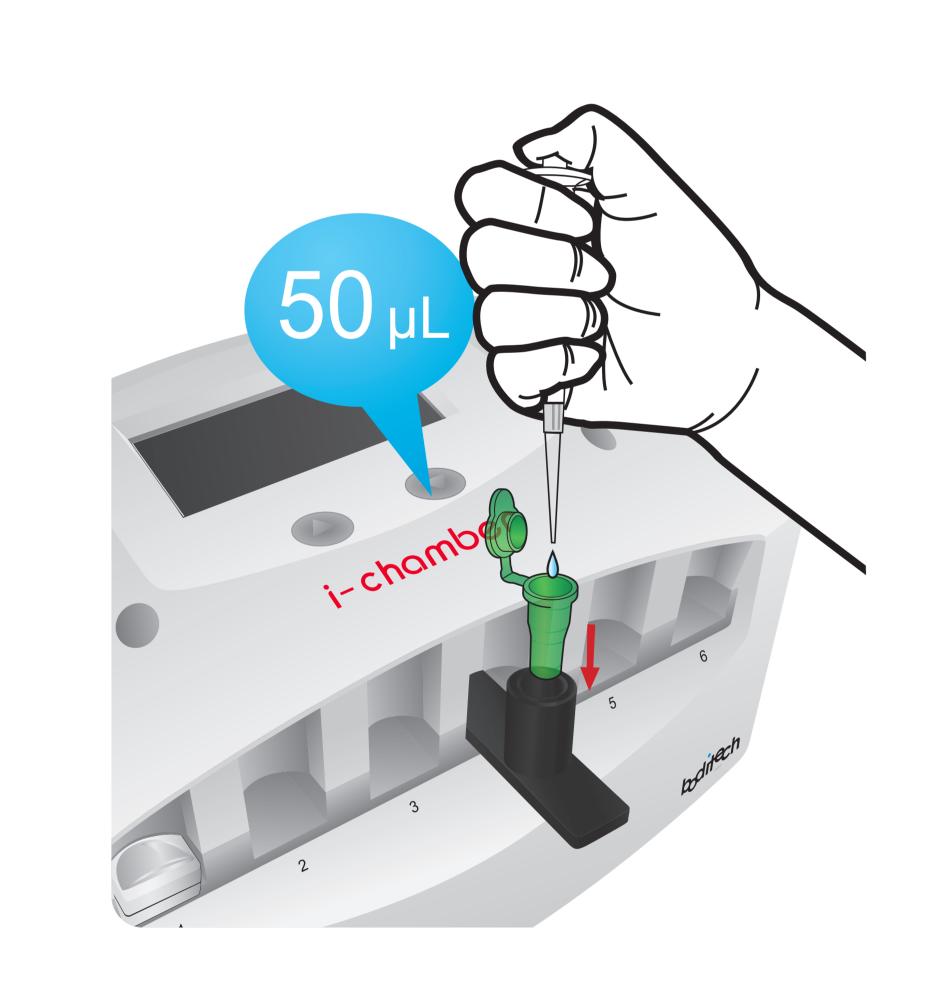


Transfer 50uL of releasing buffer into Sample mixing tube

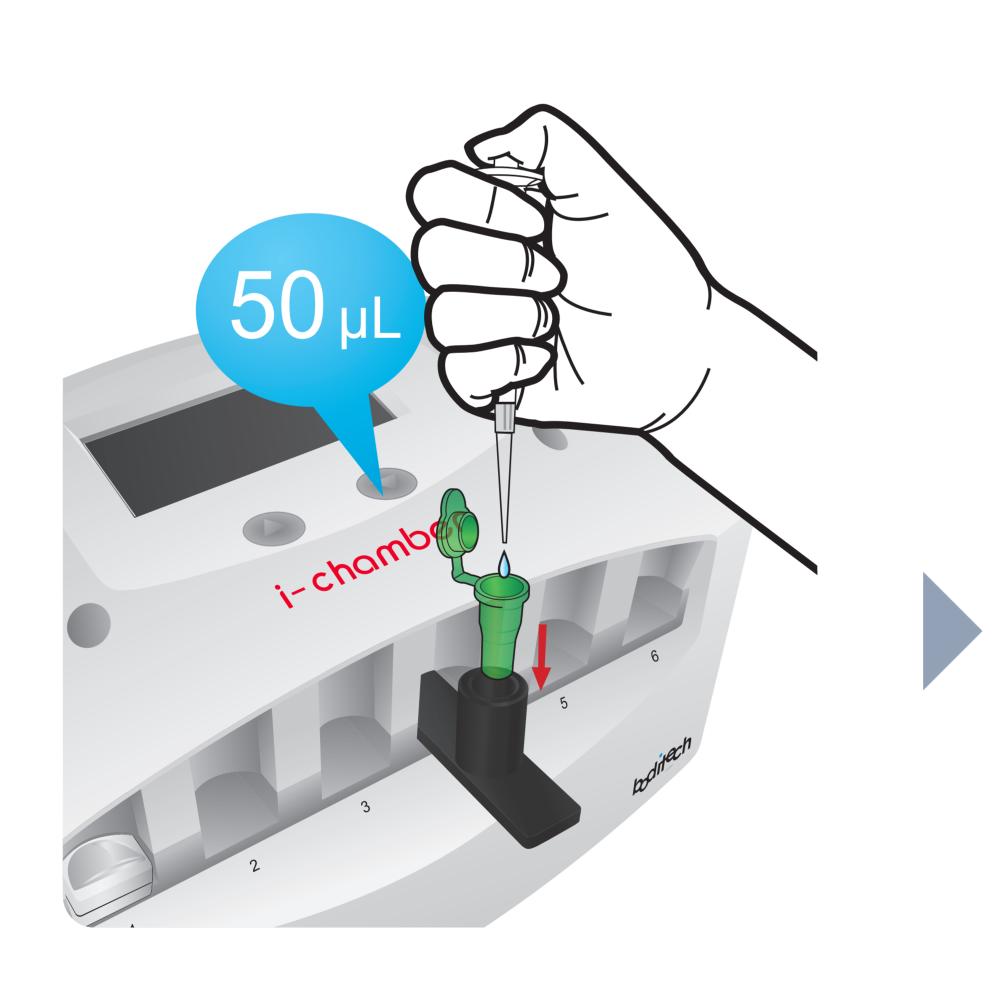

Add 50uL of sample into Sample mixing tube.

III. Load and Read

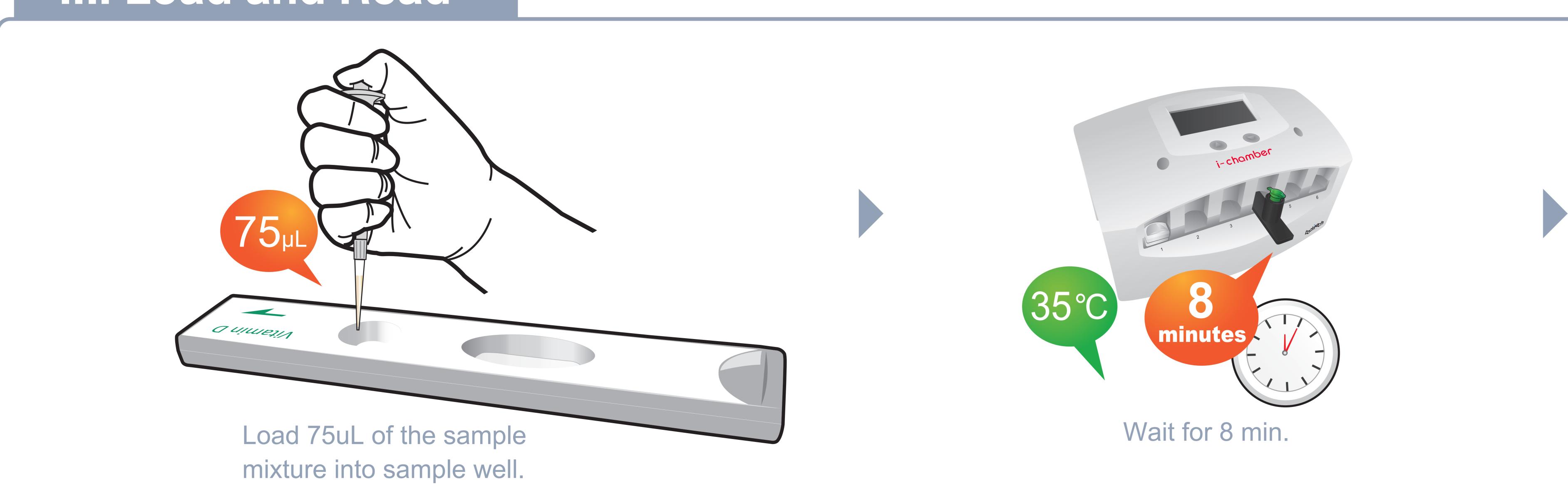




For technical assistance; please contact:Tel: +82-33-243-1400. Fax: +82-33-243-9373Boditech Med Inc.' Technical ServicesE-mail: sales@boditech.co.kr www.boditech.co.kr


ichromodTM Vitamin D Quick Reference Guide

i-chamber




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Setup i-chamber at 35°C



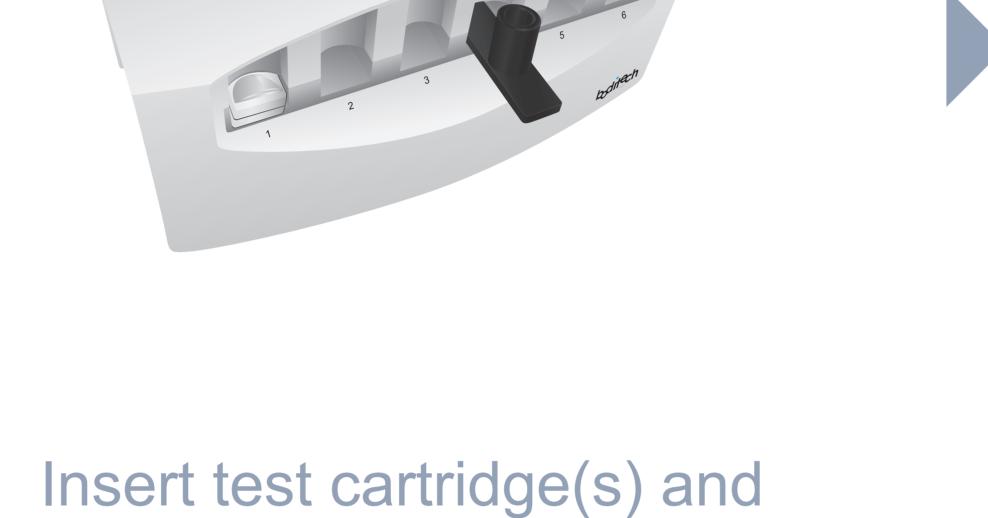



inserting tube block(s) into i-chamber

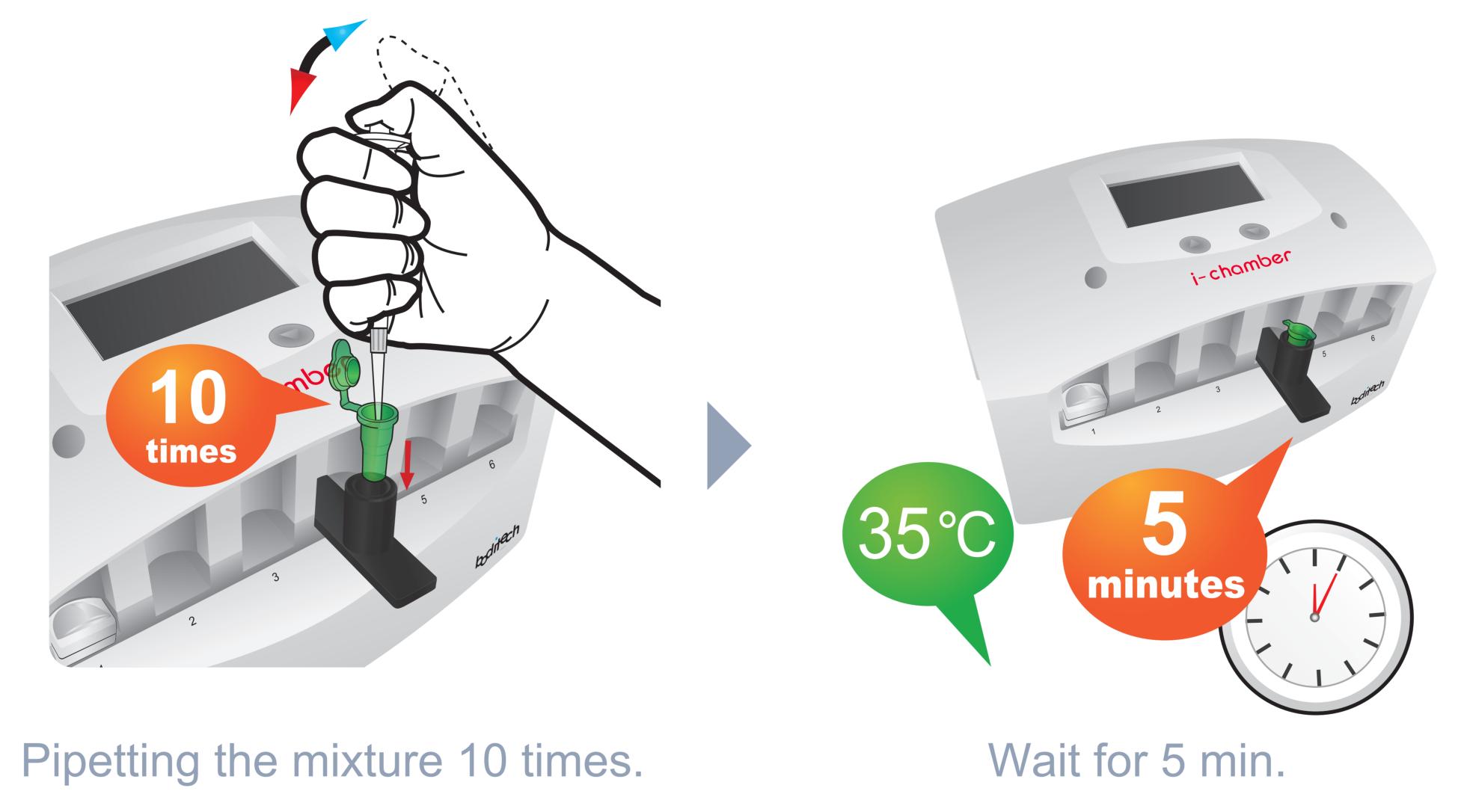




Insert sample mixing tube into the inserting tube block. Wait for 10min.

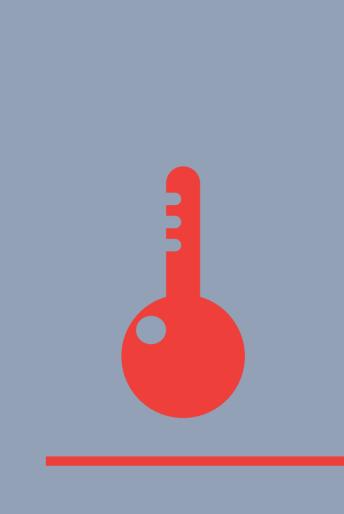


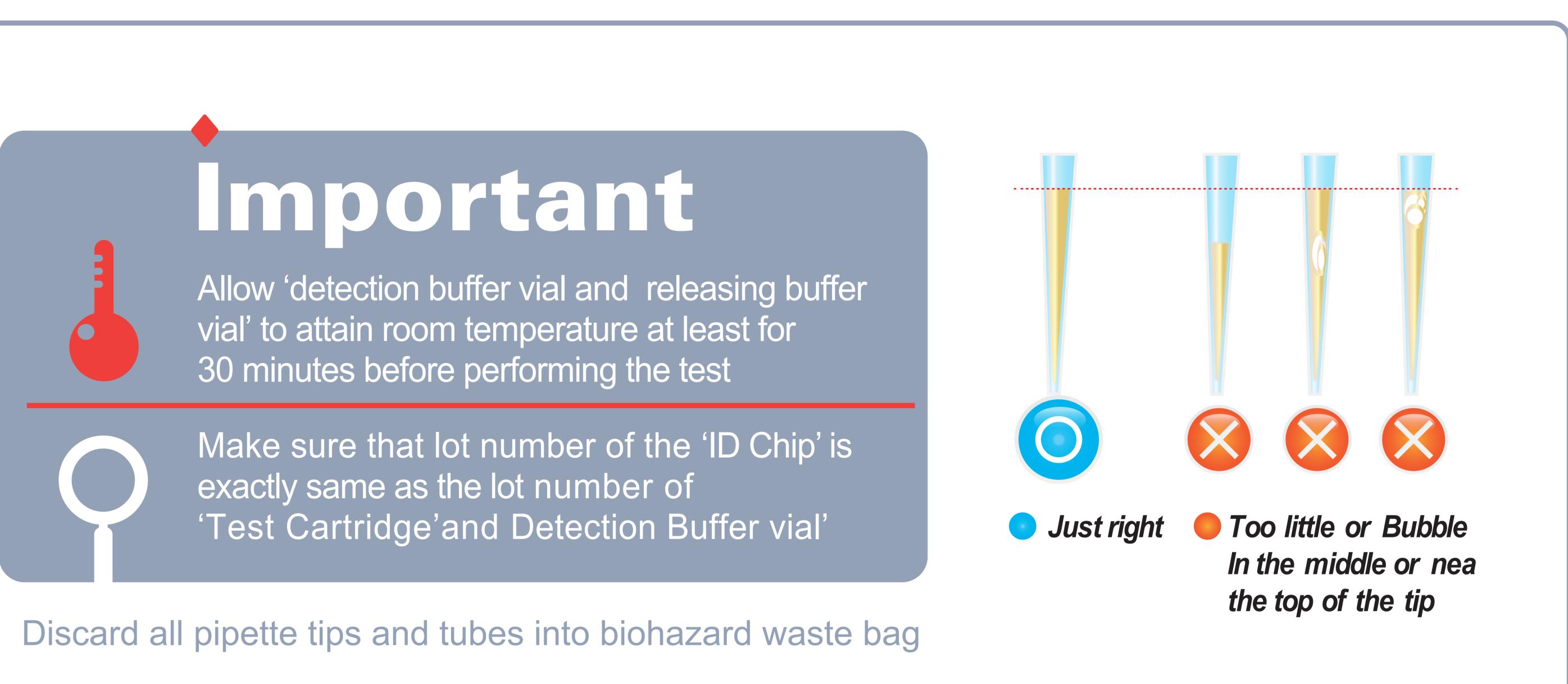




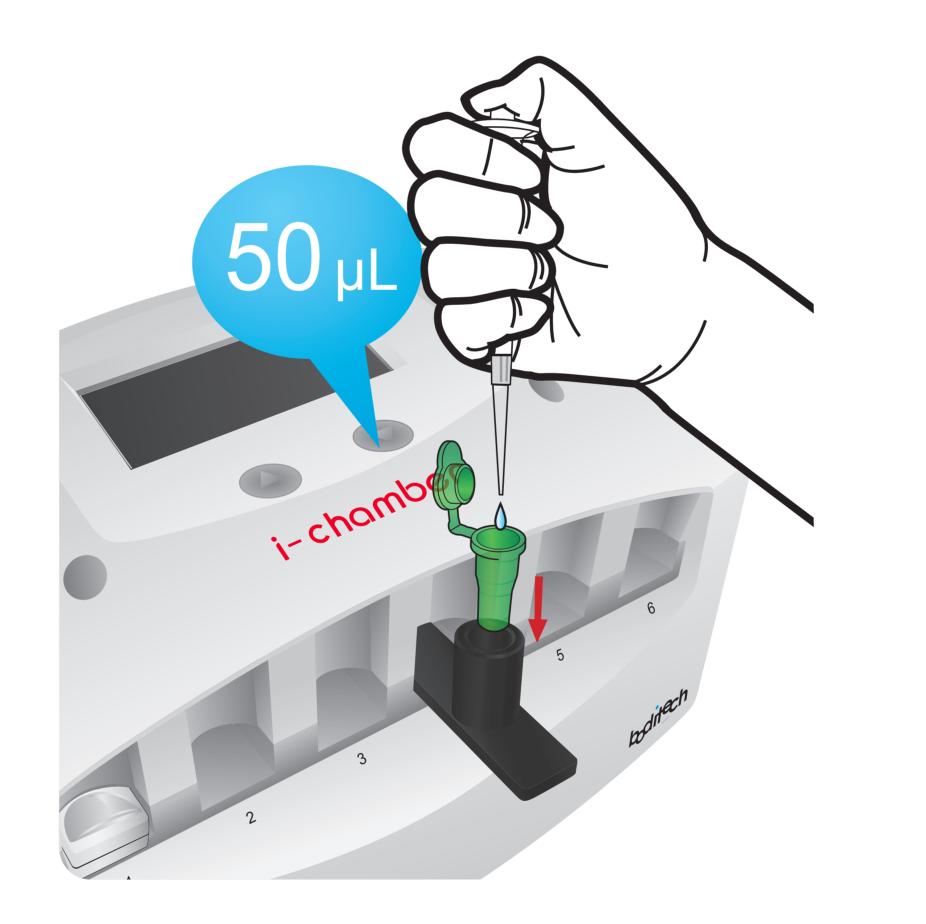

Add 100uL of detection buffer into sample mixing tube.

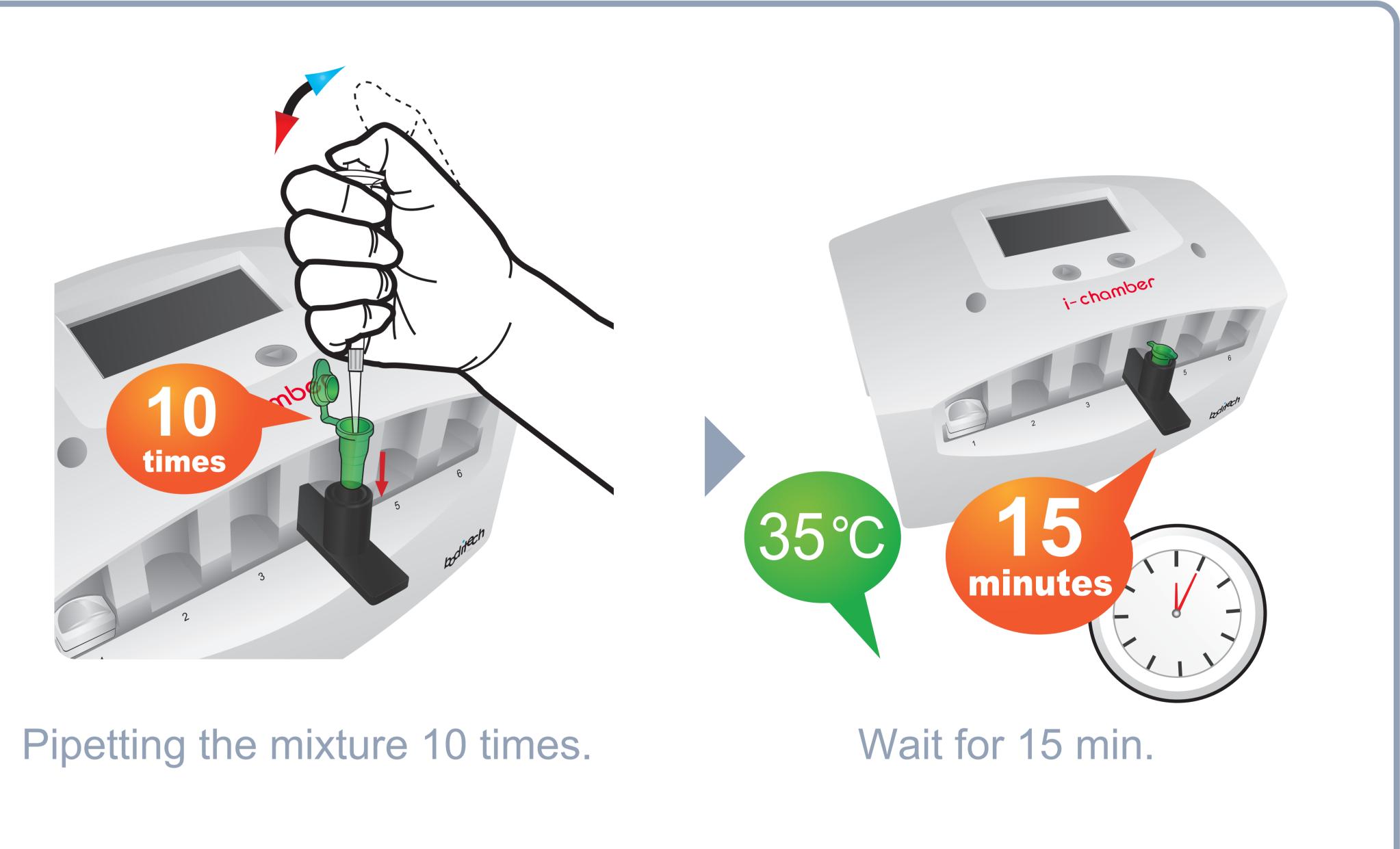

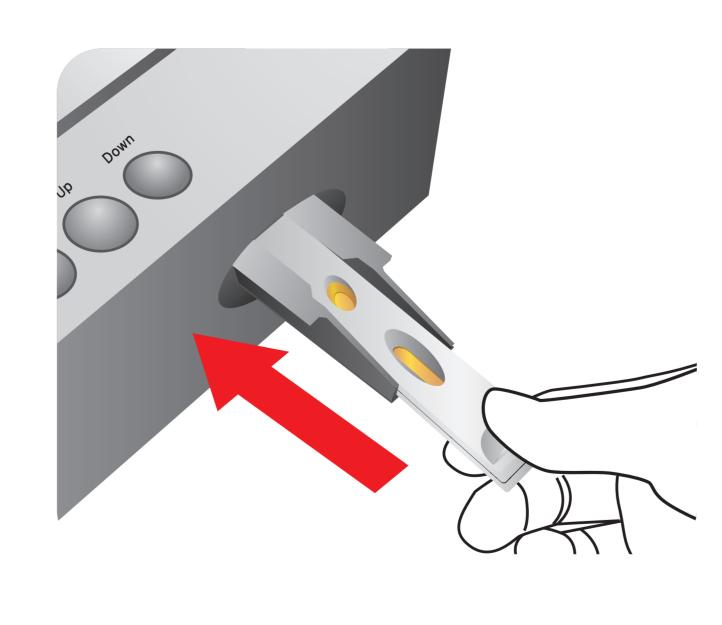

Insert the test cartridge into ichroma reader and press 'SELECT' button

This is not a complete Package Insert. For more detailed instructions, please refer to package insert.

