

Test Components



Test Cartridge



Detector tube (Granule)



Detector diluent



ID Chip



i-chamber



Caution







25°C

Before the test, set the temperature at 25°C

Test Procedure

- 1** Draw 300µL (Detector diluent)
- 2** Add it into Detector tube
- 3** Draw 75 µL (Serum / Plasma / Control)
- 4** Add it into Detector tube
- 5** Pipetting the mixture 10 times.
- 6** Shake 10 times
- 7** Wait 8 minutes

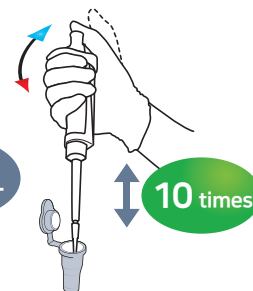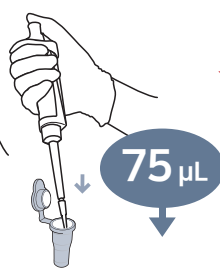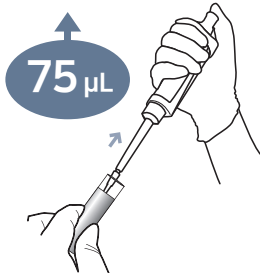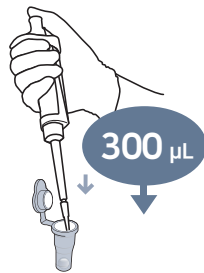

- 8** Draw 75 µL
- 9** Load the sample mixture
- 10** Wait 8 minutes
- 11** Insert the test cartridge
- 12** Press 'Start'
- 13** Read the test result

