



## II. Mix Sample

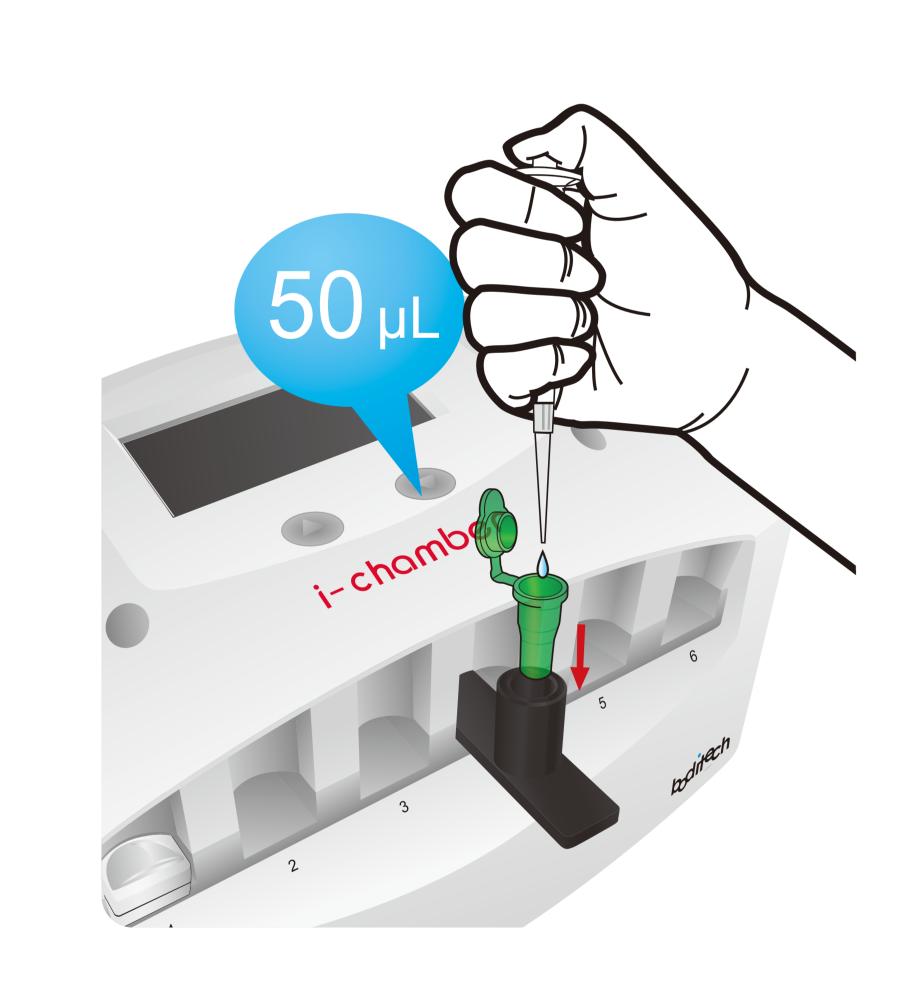


Transfer 50uL of releasing buffer into Sample mixing tube

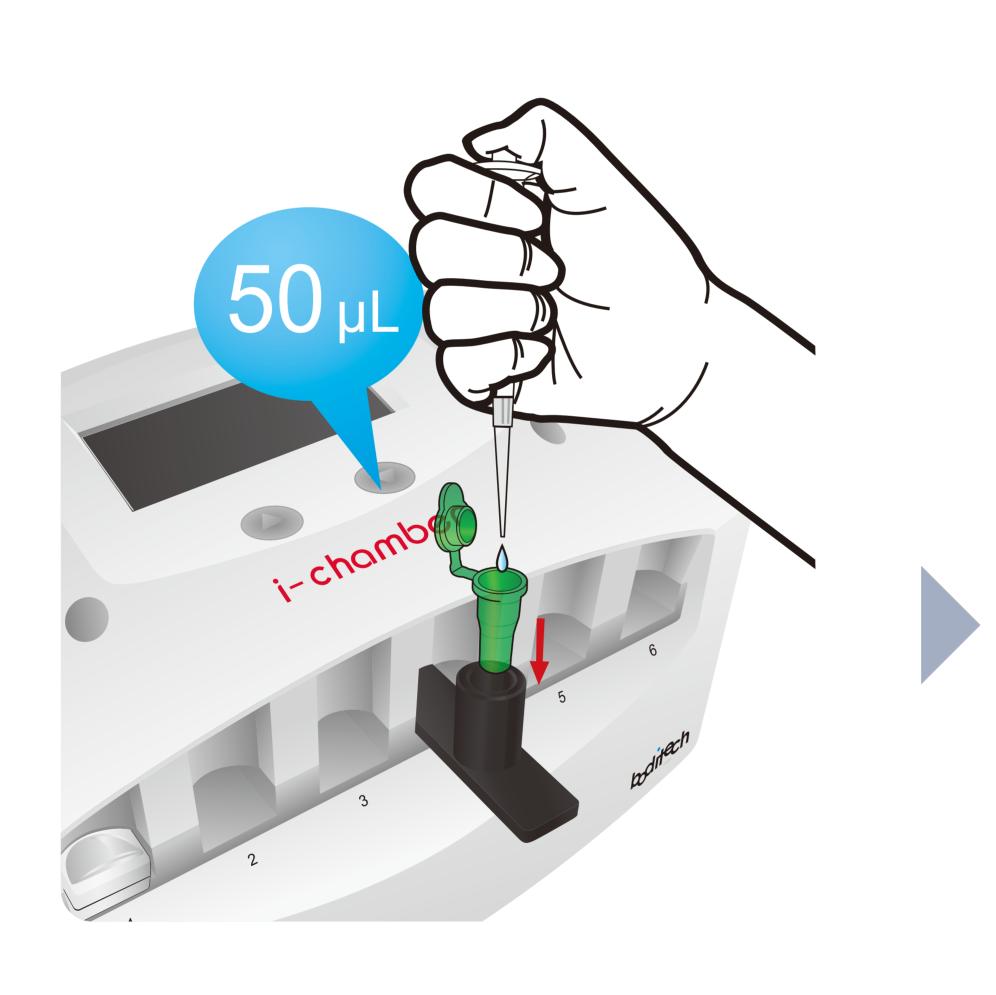

Add 50uL of sample into Sample mixing tube.

## III. Load and Read

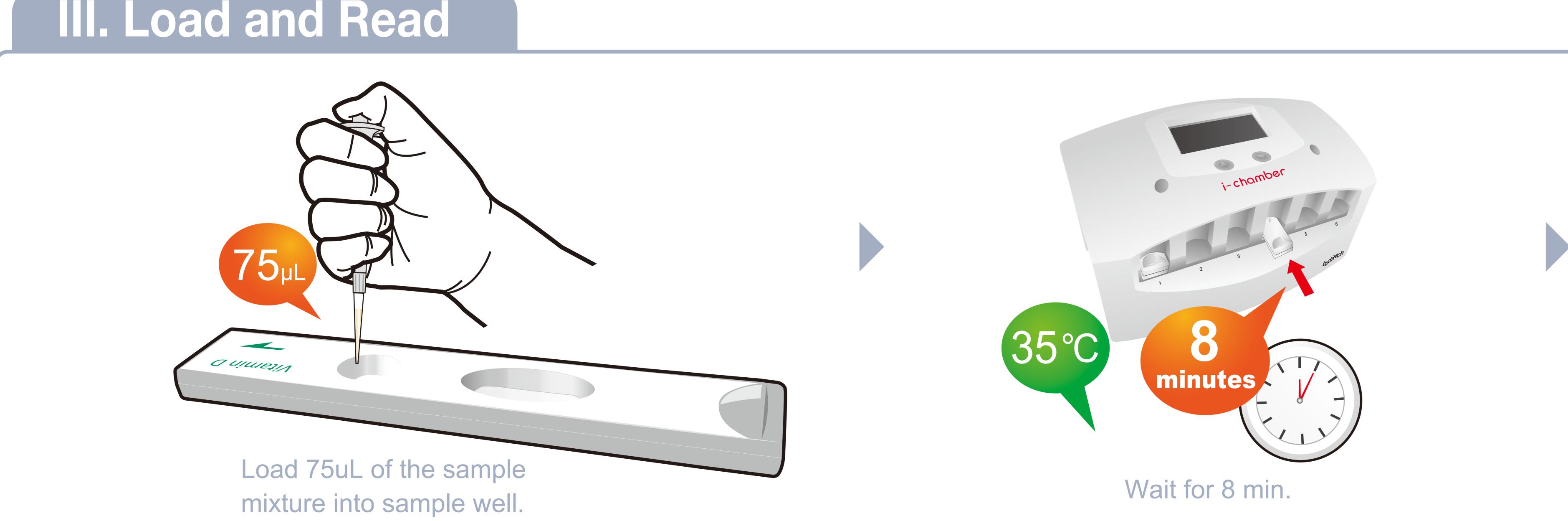




For technical assistance; please contact: Tel: +82-33-243-1400 Boditech Med Inc.' Technical Services Fax: +82-33-243-9373 www.boditech.co.kr

## ichromod<sup>TM</sup> Vitamin D Quick Reference Guide




Insert test cartridge(s) and inserting tube block(s) into i-chamber

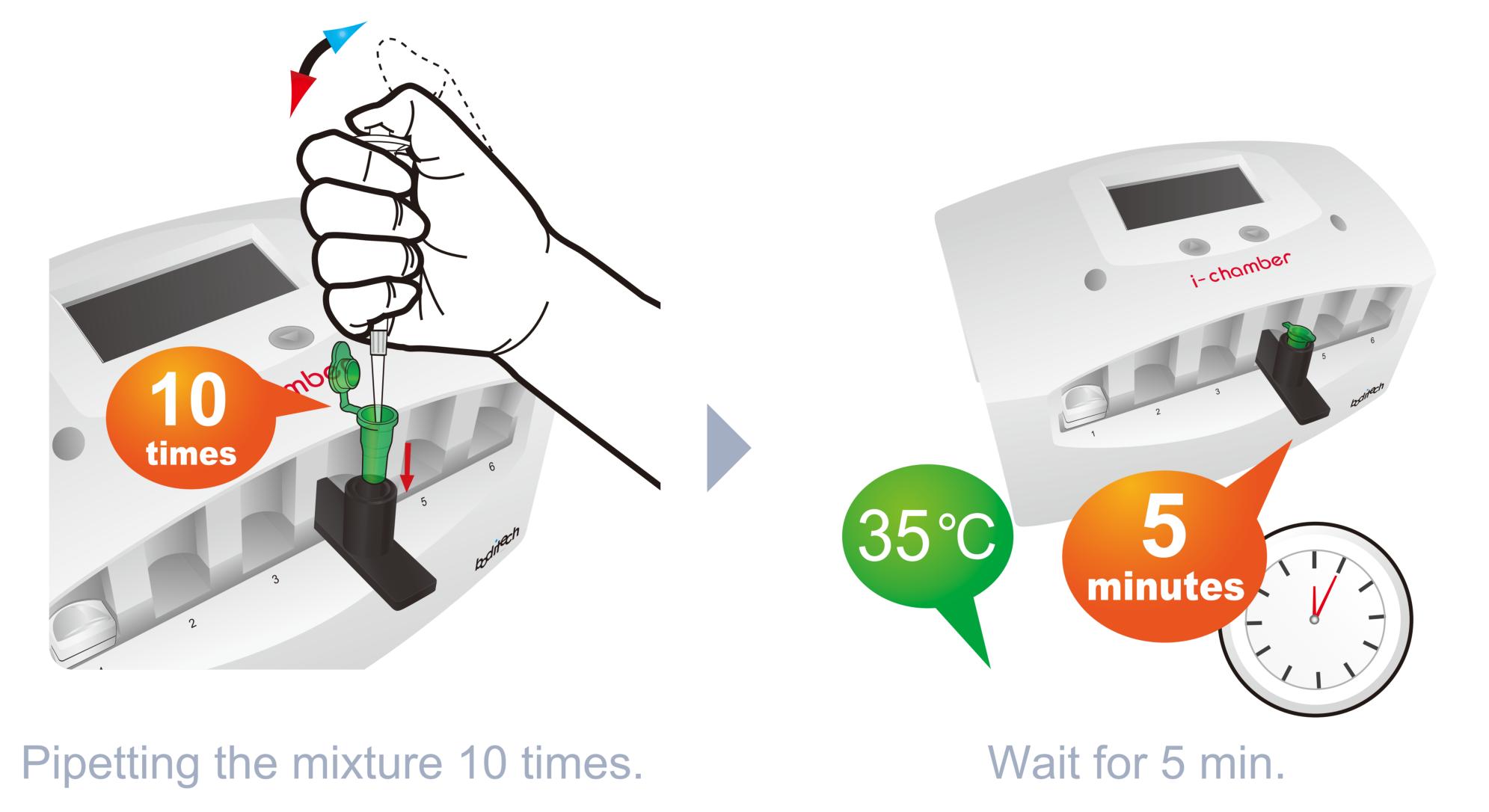

E-mail: sales@boditech.co.kr




into the inserting tube block. Wait for 10min.

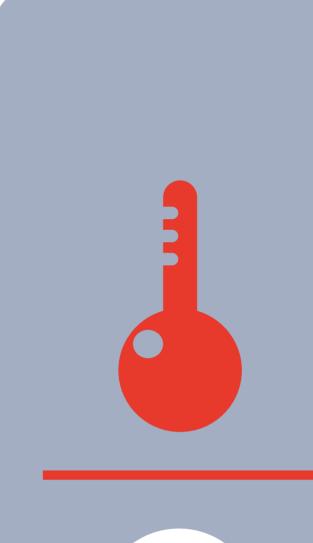

## mportant

vial' to attain room temperature at least for 30 minutes before performing the test

Discard all pipette tips and tubes into biohazard waste bag

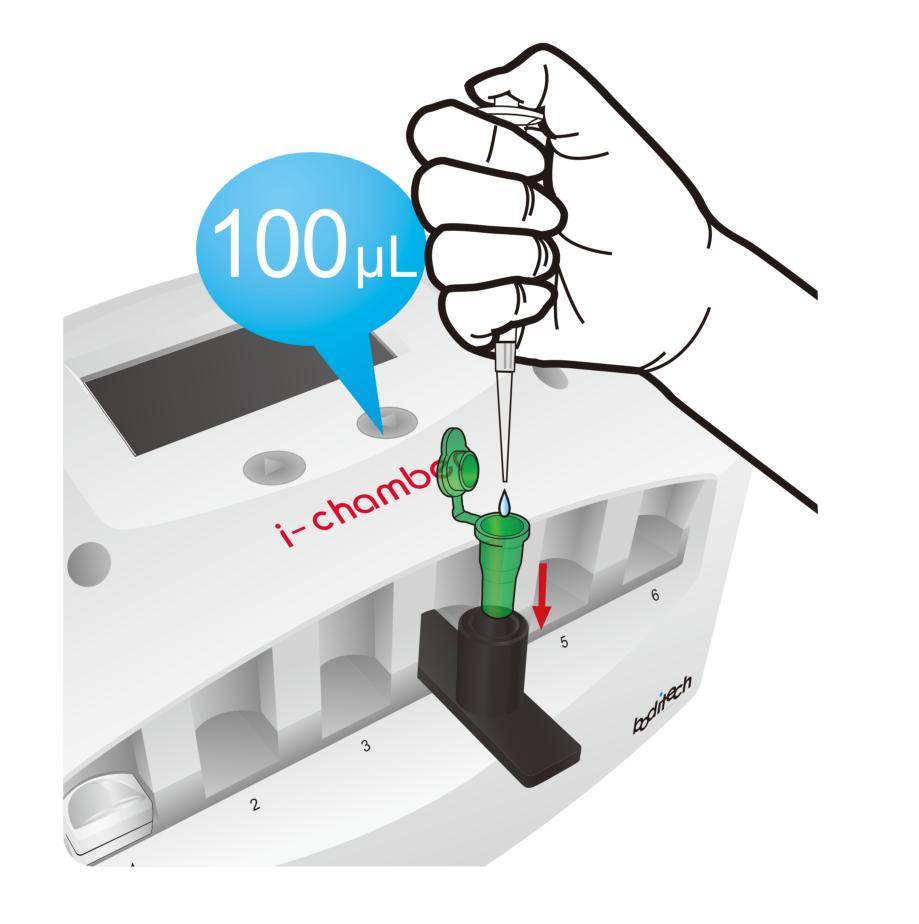

Add 100uL of detection buffer into sample mixing tube.







